

Guía de colores VITA® Classical organizado para grupos de color →		A0,5			A1			A2			A3			A3,5			A4			B1			B2			B3			B4			C1			C2			C3			C4			D2			D3			D4						
Diente Área →		Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	
		Tonos ASTERIA ↓																																																						
<b>A1B</b>	E				B	E													B	E																																				
<b>A2B</b>					E		B	E											E		B	E							B	E	B	E										B	E													
<b>A3B</b>							E		B	E																	E		E		B	B										E		B	E	B	B									
<b>A3,5B</b>									E		B	E																						B		B	E							E		B										
<b>A4B</b>											E	E	E																							E																				
<b>BL</b>	E																																																							
<b>B3B</b>																																																E	E							
<b>NE</b>		B		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E										
<b>OcE</b>																																																								
<b>WE</b>		E		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B										
<b>TE</b>																																																								
<b>YE</b>										B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B										

Guía de colores VITA® Classical organizado por brillo (valor) →		A0,5			B1			A1			B2			D2			A2			C1			C2			D4			A3			D3			B3			A3,5			B4			C3			A4			C4						
Diente Área →		Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	Cervical	Body	Incisal	
		Tonos ASTERIA ↓																																																						
<b>A1B</b>	E				B	E	B	E																																																
<b>A2B</b>					E		E		B	E	B	E	B	E	B	E	B	E	B	E	B	E																																		
<b>A3B</b>									E		E		E		E		E		E		E		E		B	B	B	E	B	E	B	E	B	B							B	B														
<b>A3,5B</b>																																																								
<b>A4B</b>																																																								
<b>BL</b>	E																																																							
<b>B3B</b>																																																								
<b>NE</b>		B		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E		E										
<b>OcE</b>																																																								
<b>WE</b>		E		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B		B										
<b>TE</b>																																																								
<b>YE</b>																																																								