ESTELITE ASTERIA Custom Shade Guide

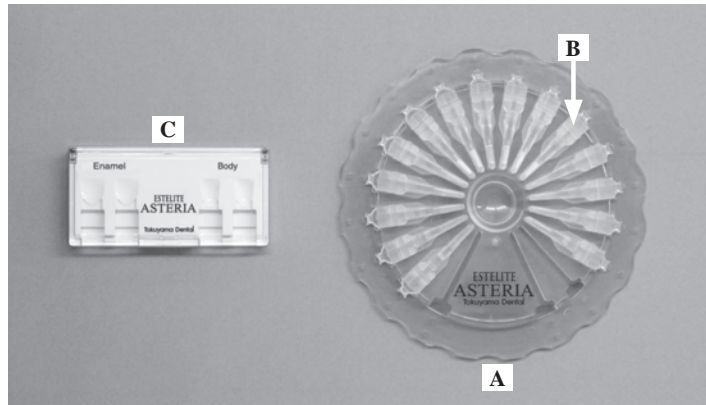
Fig. 1

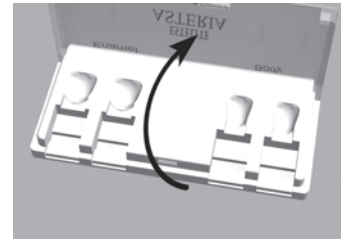
Fig. 2

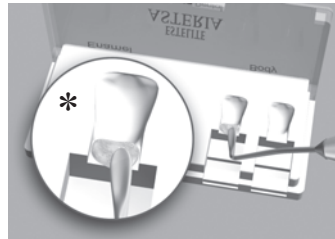
Fig. 3

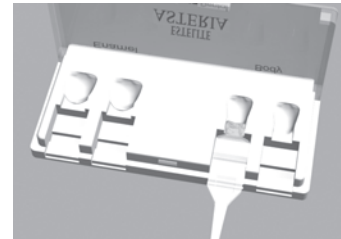
Fig. 4

Fig. 5

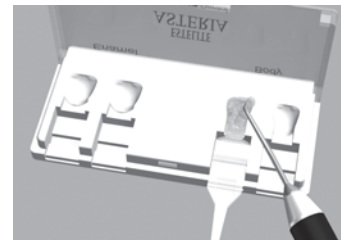
Fig. 6

Fig. 7

Fig. 8

Fig. 9

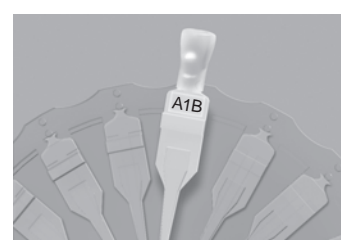
Fig. 10

ENGLISH

ESTELITE ASTERIA Custom Shade Guide is a shade guide kit for making your own custom shade guides. ESTELITE ASTERIA Custom Shade Guide consists of the following items.

Item	Material	Quantity
A. Holder	ABS (Acrylonitrile-Butadiene-Styrene) resin	1
B. Shade Guide Handle	Polypropylene	15
C. Mold	ABS (Acrylonitrile-Butadiene-Styrene) resin	1

HOW TO USE

- Fig. 1. Select the Mold.
- The Mold has two types of tooth shapes. The left side molds are wide and shallow. These are suitable for Enamel shades. The right side molds are narrow and deep. These are suitable for Body shades.
- Fig. 2. Flip up the cover cap of the Mold.
- Fig. 3. Before setting the Shade Guide Handle into the appropriate slot, fill the root portion* of tooth shaped mold with ESTELITE ASTERIA resin paste using an instrument to anchor the Handle to the resin.

- Fig. 4. Insert the Handle with its flat side up until it seats in completely and is flush with the Mold surface.
- Fig. 5. Fill up the remaining portion of the tooth Mold with ESTELITE ASTERIA resin paste.
- Fig. 6. Remove excess paste and flatten the surface.

- Fig. 7. Close the cover cap and press down firmly to squash the resin paste.
- Fig. 8. Light irradiate for 10 sec. (A4B shade: 20 sec.) through the cover cap.
- Fig. 9. Re-open the cover cap and remove the Handle from the Mold. Trim side resin flashes, as needed.
- Fig. 10. Set the finished Handle (Individual Shade) in the Holder.

NOTICE FOR USE OF THE ESTELITE ASTERIA CUSTOM SHADE GUIDE

1. Keep the ESTELITE ASTERIA Custom Shade Guide away from direct sunlight to prevent a change in color.
2. Refer to the following table when cleaning the ESTELITE ASTERIA Custom Shade Guide.

	Cleaning method
Correct	- wash with soap and water - ultrasonic cleaning while immersed in water - immerse in a 3% glutaraldehyde solution - wipe with a cotton swab soaked in an alcohol (70%) solution
Incorrect	- autoclave ¹⁾ - dry heat sterilization ¹⁾ - immerse in an alcohol (70%) solution ²⁾ - immerse in a sodium hypochlorite solution ³⁾ - immerse in a hydrogen peroxide solution ³⁾

- 1) Shade Guide Handle, Holder and Mold will melt.
- 2) Holder and Mold will whiten and crack.
- 3) Individual Shade may discolor.

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Tokuyama Dental Corporation
Tokuyama Dental Italy S.r.l.